FR - Instructions de montage
 GB - Assembly instructions
 ES - Instrucciones de instalación
 IT - Istruzioni di montaggio
 DE - Montageanleitung
 NL - Montage-instructie
 PT - Instruções de instalação
 HR - Upute za montazu


MSV FRANCE®

European Bathroom Accessories Manufacturer

MSV FRANCE®

European Bathroom Accessories Manufacturer

FR - **Attention** : Ne pas l'utiliser comme escabeau ou aide pour monter.
 GB - **Attention** : Do not use as a step or climbing aid.
 ES - **Atención** : No montar en la silla o no utilizar para el taburete.
 IT - **Attenzione** : Non utilizzarlo come sgabello all'aiuto per salire.
 DE - **Achtung** : Benutzen sie ihn nicht als einen Hocker oder um auf ihn zu steigen.
 NL - **Opgelet** : Niet te gebruiken als ladder of opstap.
 PT - **Precauções** : Nao usar como mocho nem ficar de pe sobre sede.
 HR - **Upozorenje** : Ne koristiti kao stuba ili pomak kod uspinjanja.

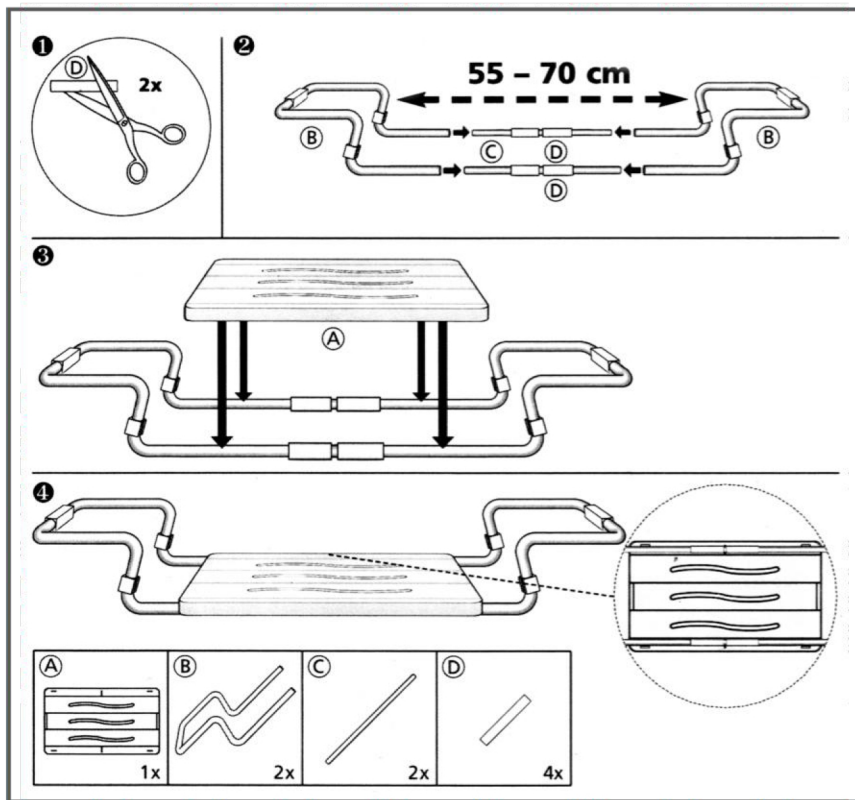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GB

General specifications :

Use the seat only to sit in the bathtub.
Mount the seat in accordance with the instructions.
We don't recommend you to remove the seat after mounting.
Do not stand up on seat. Risk of falling.
The seat carries maximum 150 kg. Do not exceed this weight.
It is not suitable for the bathtubs having less than 6 cm of interior walls.
There is no toxic (poisonous) substance in this product.

Caution :

Regularly clean the bath seat and keep clean.
Do not use abrasive sponge and products that contain chlorine or acid. Wash different parts with a product having detergent.
Rinse parts with hot water. Wipe them with help of a cloth.

Technical data :

Bathroom stool width: 55-70 cm.
Dimensions of the sitting place: 36x26 cm.
Max. load capacity: 150 kg.
Material: Aluminum + PP

Exceptional cases :

MSV Company should not be held responsible for any personal and monetary damages which have been caused or will be caused by wrong use of the product or failure to mount correctly.
Make sure that there is a nonslip mat on the shower floor and outside of the shower. You could slip on a wet floor. We cannot accept any responsibility for damage due to slippery shower floor or else.

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MENAGE SELECTION VALNET S.A.S.